

Single use lancing device; Simple 3 step testing

1 Remove the sterility cap.



- Twist and pull off the purple sterility cap.
- Do not use the device if the purple sterility cap has previously been removed.
- A built-in mechanism ensures that the lancing device can only be used once.
- Prevents accidental needlestick injury.

2 Select penetration depth.



- Accu-Chek Safe-T-Pro Plus offers 3 adjustable depth settings for different sample sizes and skin types.
- Select the desired penetration depth only after removing the sterility cap and choose the depth you require by turning the depth adjuster.
- Depth settings:*
 1. Low depth (1.3mm)
 2. Medium depth (1.8mm)
 3. High depth (2.3mm)

3 Obtain blood.



- Press the Accu-Chek Safe-T-Pro Plus lancing device firmly against the puncture site.
- Push the release button down*.
- Collect blood.

* Trigger button resistance - firm pressure required for needle to penetrate skin, this prevents accidental needle fire.

* Please check your local guidelines for patient depth settings

For further assistance please contact your local

Accu-Chek Enquiry Line: 1800 25 18 16 (Australia)

Accu-Chek Enquiry Line: 0800 80 22 99 (New Zealand)

www.accu-chek.com.au
Roche Diagnostics Australia Pty Ltd
31 Victoria Avenue, Castle Hill NSW 2154
ABN 29 003 001 205
Accu-Chek Enquiry Line: **1800 25 18 16**

www.accu-chek.co.nz
Roche Diagnostics
15 Rakino Way, PO Box 62 089, Mt Wellington, Auckland, New Zealand
Accu-Chek Enquiry Line: **0800 80 22 99**
Always read the label and follow the manufacturer's instructions

Accu-Chek Safe-T-Pro-Plus for convenience and comfort.

ACCU-CHEK®